


How to operate gateway?



GW9014A
TeleHealth Gateway



Quick Installation Steps
Distributed by
Fora Care Inc.
810 Lawrence Drive, Suite 104,
Newberry Park, CA 91320 USA
www.foracare.com/usa

Your Package Contains

- Data Cable **A**
- AC Power Adapter **B**
- (Optional) Telephone Line **C**
- (Optional) Network Cable **D**

STEP 1 • Rear View of Meter •

Insert A into port 1



STEP 2

If you are using Ethernet, connect B to Port 2. Then connect it into your Internet router.



STEP 3

If you are using a Phone, connect C into Port 3. Then connect C into the phone wall jack.

To use your phone, plug your phone's line into Port 4.



STEP 4

Insert D into Port 5. Plug D into an outlet.

Switch on the Power Button



Connecting your monitor to your Gateway

Connect A into your monitor.



Your monitor will read "MC" once it is connected.

Available in English
and Spanish.